Update WheelDrive april 2023

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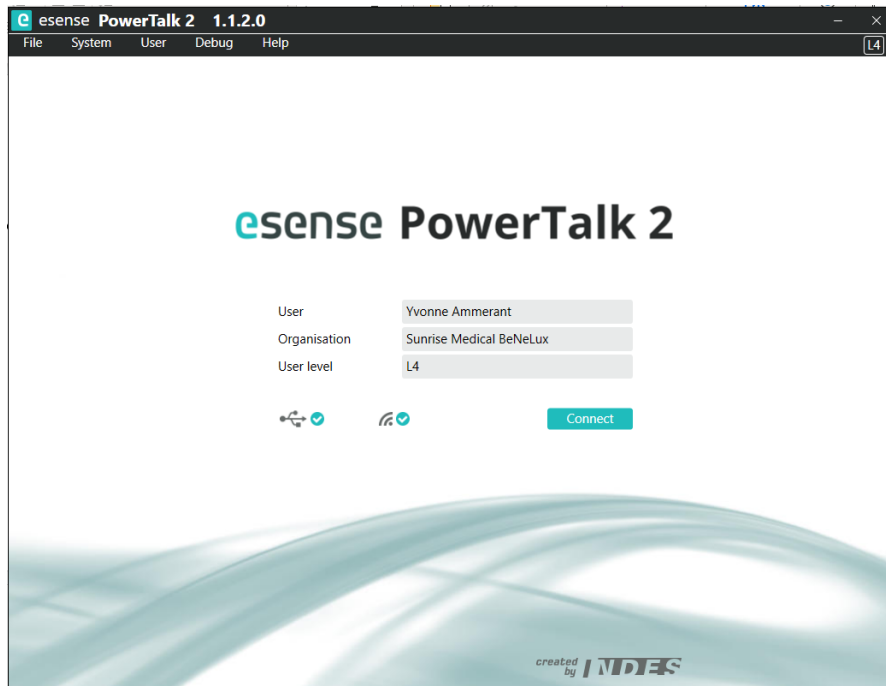
Update firmware

The screenshot shows the esense PowerTalk 2 1.1.2.0 software interface. The top menu bar includes File, System, User, Debug, and Help. Below the menu is a toolbar with icons for Connect, History, Save report, Send report, and Documentation. The main content area is divided into several sections: Overview (selected), Activity, Usage, Health, Values, Settings, Counters, Log, and Terminal. The Overview section displays system information on the left, components on the right, and system time and periodic check details at the bottom. A yellow circle highlights the 'Status' column in the 'Components' table, which shows an orange exclamation mark icon, indicating a firmware update is available.

System	
System type	200 - WheelDrive
Product type	
Product serial	1902RW05324
Service ID	R05324
Production date	20-2-2019
Facility	
Location	

Components	
Controller	WD2-P1-4
User Interface	WD1-P2-4
Battery	V2.03

System time	
Date and time	18-4-2023 11:38:42

Periodic check	
Next date	20-2-2020


Force sensor	
Assist rim sensor	0
Drive rim sensor	0

Battery	
Capacity	3800 mAh
Charge	24 %
Status	Idle

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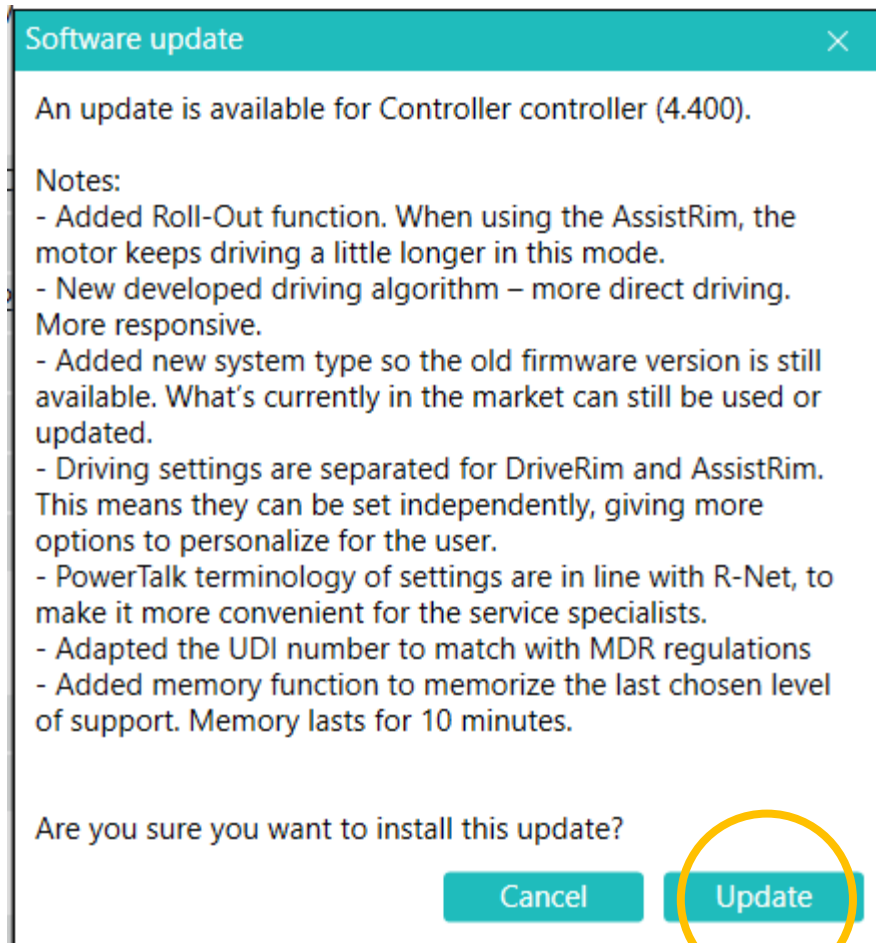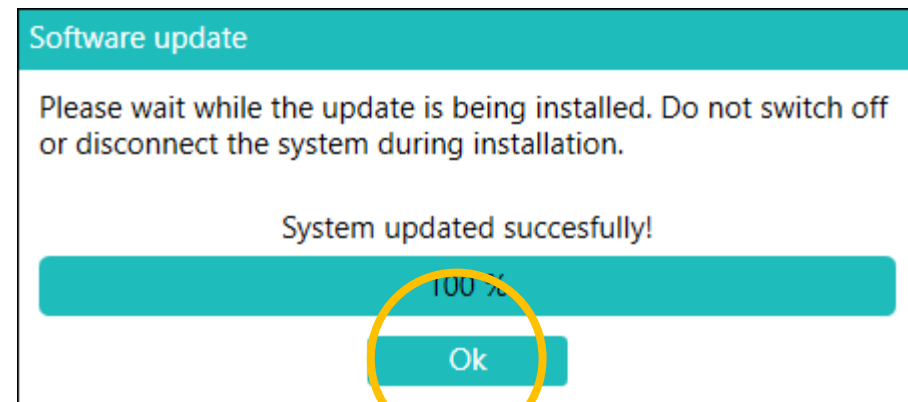
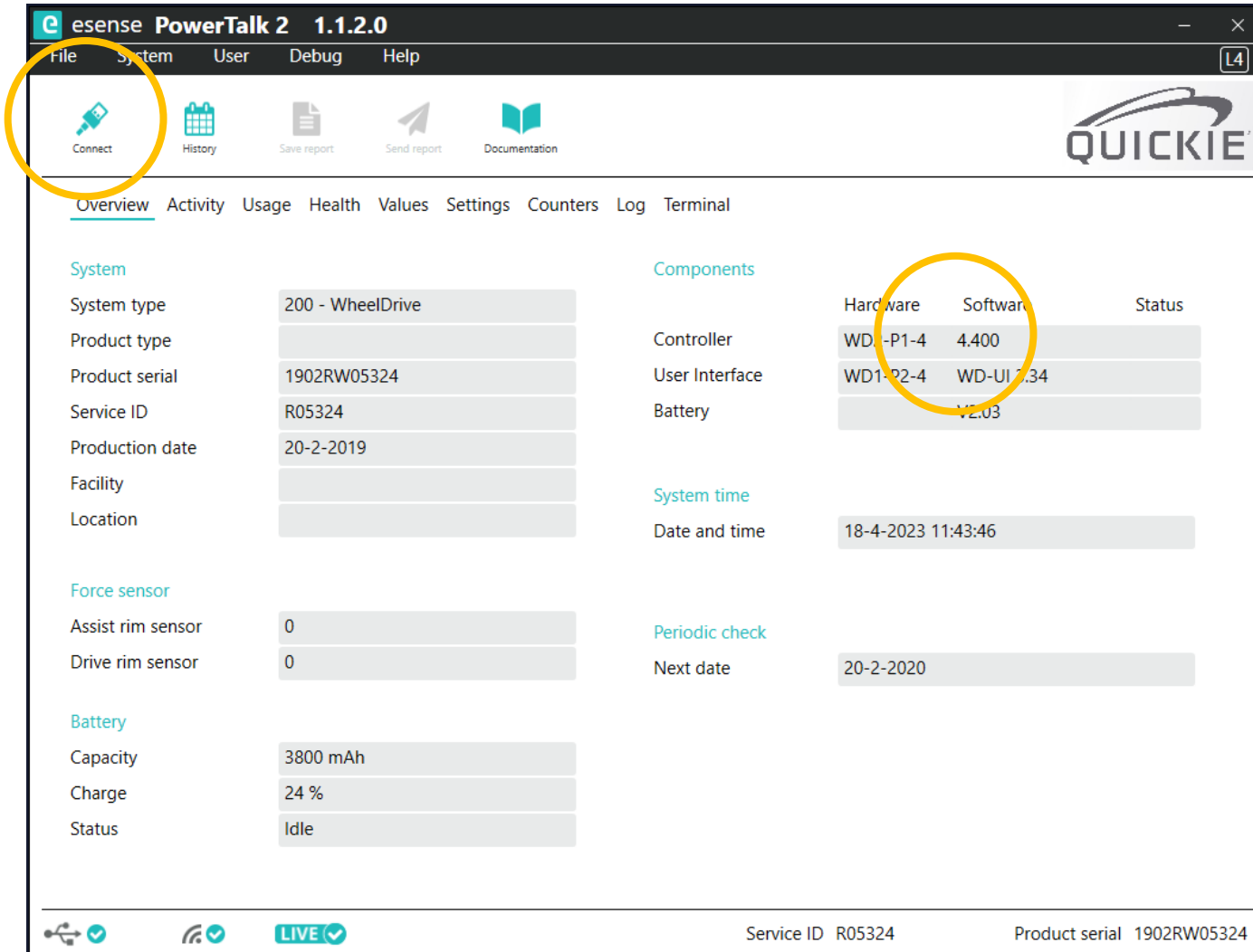
esense PowerTalk 2 1.1.2.0

File System User Debug Help

Connect History Save report Send report Documentation

Overview Activity Usage Health Values Settings Counters Log Terminal

System

System type	200 - WheelDrive
Product type	
Product serial	1902RW05324
Service ID	R05324
Production date	20-2-2019
Facility	
Location	

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
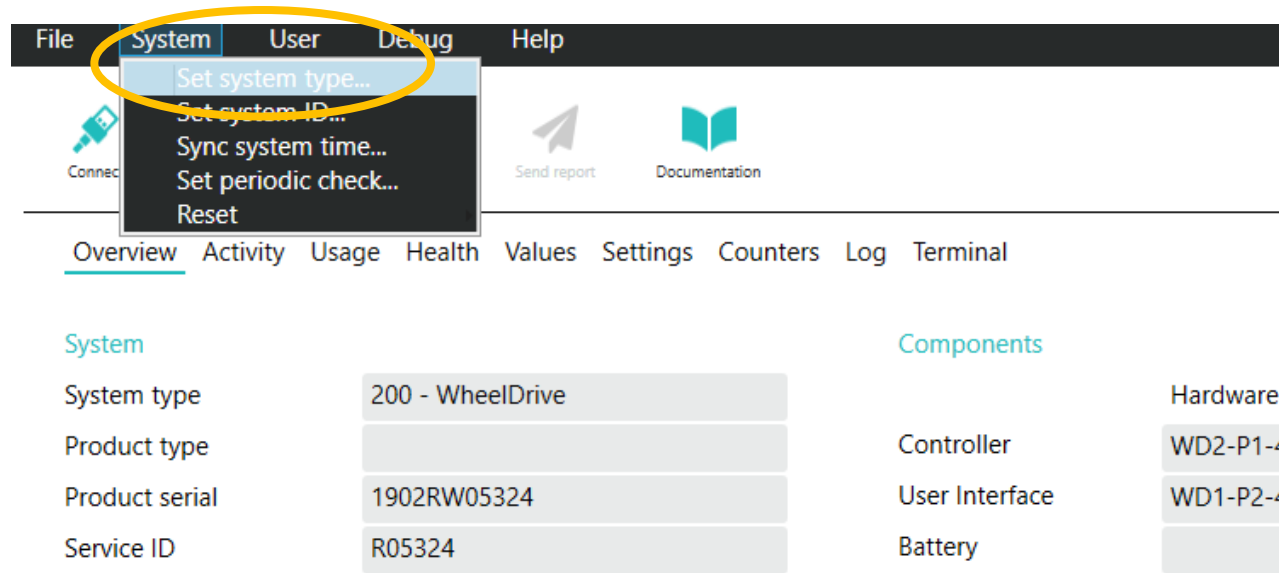
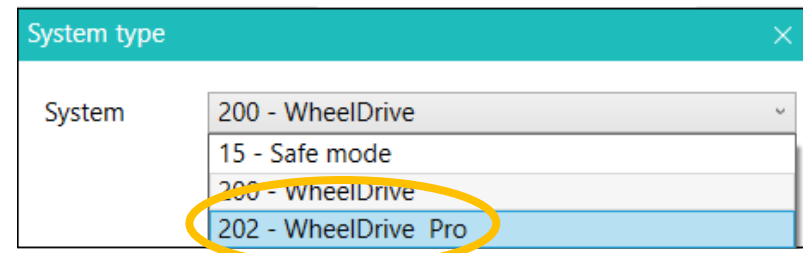
The screenshot shows the 'esense PowerTalk 2 1.1.2.0' application window. The 'Settings' tab is active, displaying a list of driving parameters under the 'Controller' section. The parameters are organized into a table with columns for the setting name, its current value, and a unit. The 'AssistRim max. forward speed' and 'AssistRim max. reverse speed' rows are highlighted with a red border, indicating they are the focus of the reset operation. The 'QUICKIE' logo is visible in the top right corner of the interface.

Controller	Setting	Info	More
Driving behaviour			
Driving settings			
Custom support level 1			
DriveRim max. forward speed	30	0,1 km/h	i
DriveRim max. reverse speed	20	0,1 km/h	i
DriveRim max. acceleration	40	%	i
DriveRim max. deceleration	30	%	i
AssistRim max. forward speed	3	0,1 km/h	i
AssistRim max. reverse speed	3	0,1 km/h	i
AssistRim max. acceleration	37	%	i
AssistRim max. deceleration	30	%	i
Custom support level 2			
Custom support level 3			
General settings			
System info			
Battery			

Buttons at the bottom: Undo changes, Write, Save settings, Load settings.

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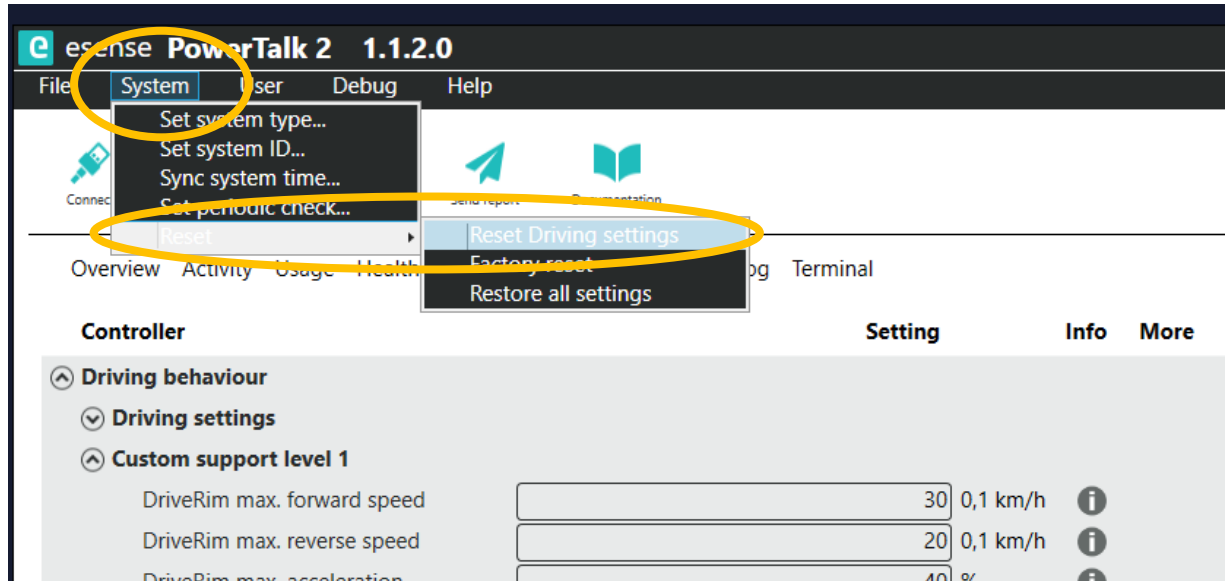
Nieuwe setting - Support level memory

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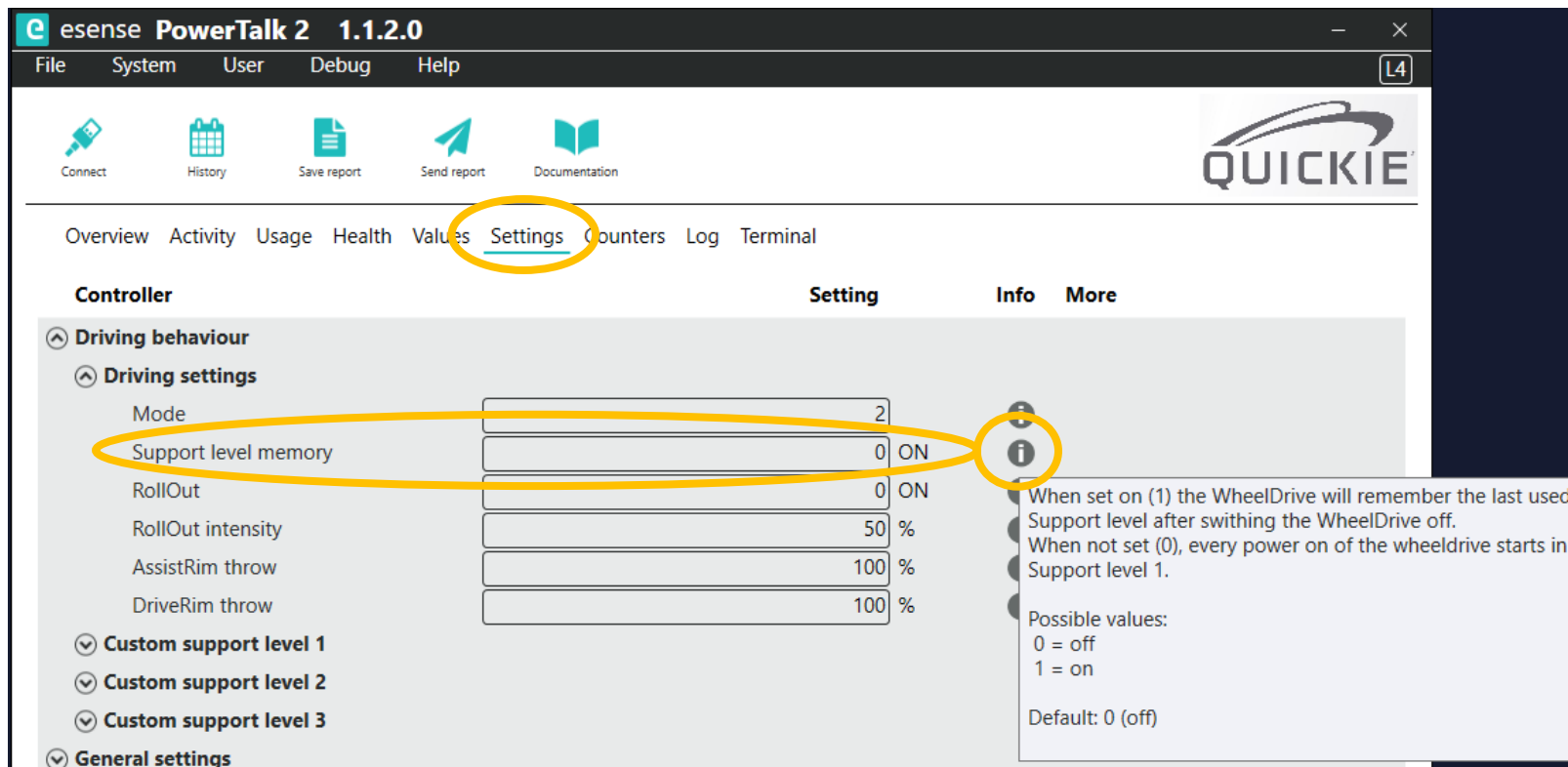The screenshot shows the esense PowerTalk 2 1.1.2.0 software interface. The 'Settings' tab is selected and highlighted with a yellow circle. Under the 'Controller' section, the 'Driving behaviour' category is expanded, and 'Support level memory' is highlighted with a yellow circle. The 'Support level memory' setting is currently set to '0 ON'. A yellow circle also highlights the information icon (i) next to the setting. A tooltip is visible on the right side of the interface, providing details about the 'Support level memory' setting.

Setting	Value	Unit
Mode	2	
Support level memory	0 ON	
RollOut	0 ON	
RollOut intensity	50	%
AssistRim throw	100	%
DriveRim throw	100	%

When set on (1) the WheelDrive will remember the last used Support level after swithing the WheelDrive off.
When not set (0), every power on of the wheeldrive starts in Support level 1.

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The screenshot shows the esense PowerTalk 2 1.1.2.0 software interface. The 'Settings' tab is selected and highlighted with a yellow circle. Under the 'Controller' section, the 'Driving behaviour' category is expanded, and the 'Driving settings' sub-category is also expanded. The 'RollOut' setting is highlighted with a yellow oval. It is currently set to '0 ON'. To the right of the settings table, an information pop-up is visible, also highlighted with a yellow circle. The pop-up contains the following text:

When enabled (1) the RollOut is active. With the RollOut enabled, the WheelDrive will roll out further when using the AssistRim.

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
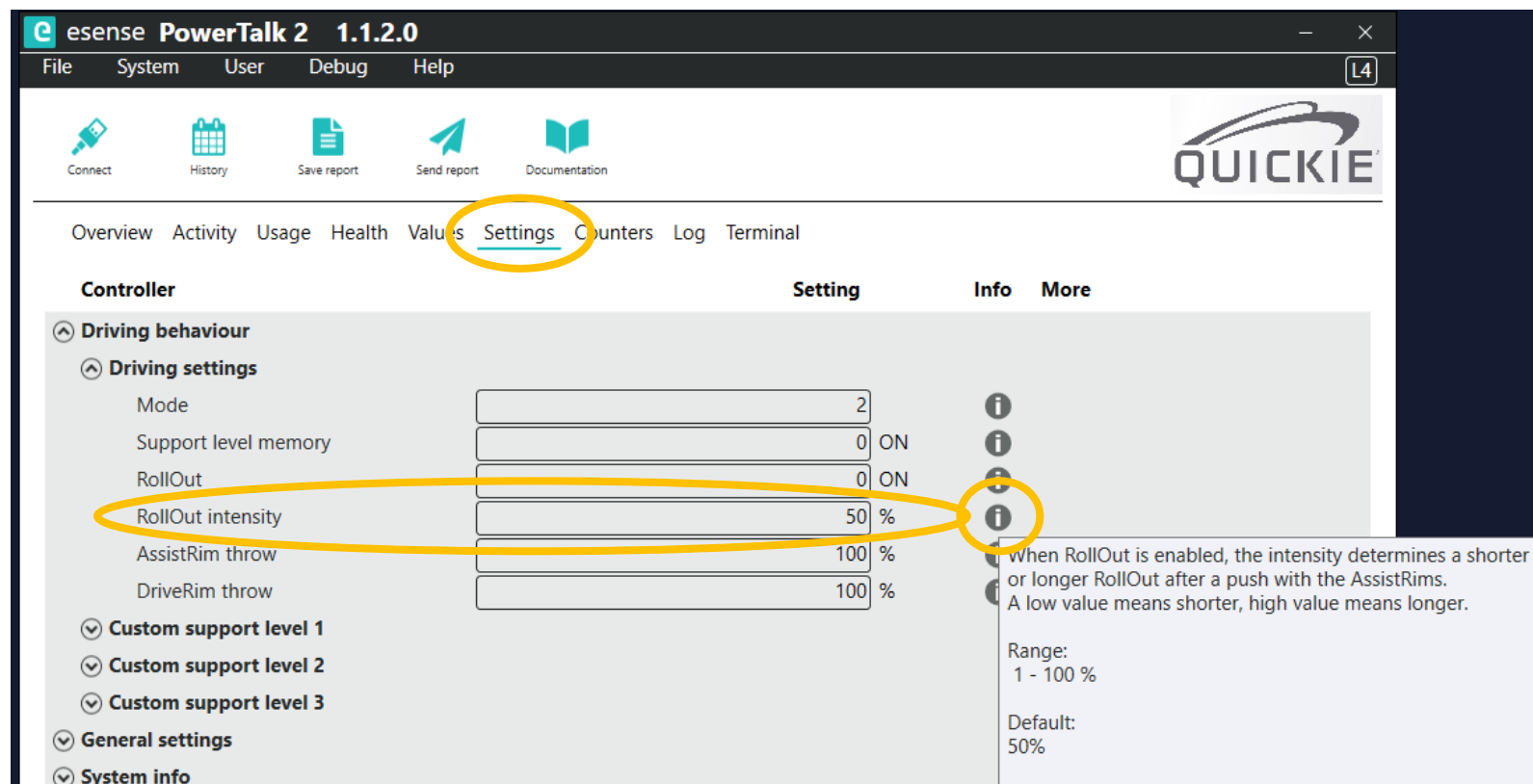
Nieuwe setting – Throw (assist & drive hoepel)

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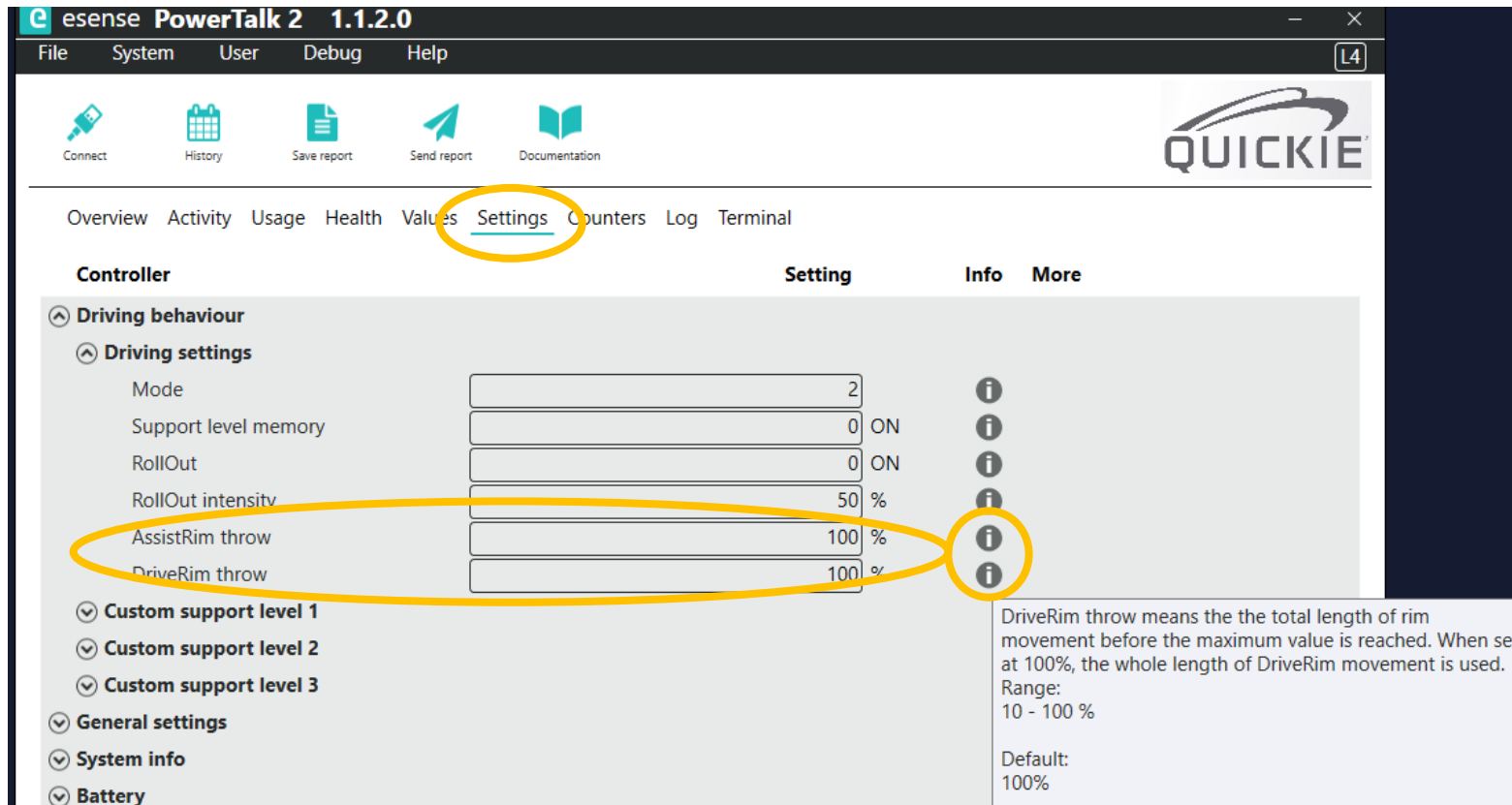AssistRim throw means the the total length of rim movement before the maximum value is reached. When set at 100%, the whole length of AssistRim movement is used.

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esense PowerTalk 2 1.1.2.0

File System User Debug Help

Connect History Save report Send report Documentation

Overview Activity Usage Health Values Settings Counters Log Terminal

Controller	Setting	Info	More
RollOut intensity	50 %	i	
AssistRim throw	100 %	i	
DriveRim throw	100 %	i	
Custom support level 1			
DriveRim max. forward speed	30 0,1 km/h	i	
DriveRim max. reverse speed	20 0,1 km/h	i	
DriveRim max. acceleration	40 %	i	
DriveRim max. deceleration	30 %	i	
AssistRim max. forward speed	30 0,1 km/h	i	
AssistRim max. reverse speed	20 0,1 km/h	i	
AssistRim max. acceleration	37 %	i	
AssistRim max. deceleration	30 %	i	
Custom support level 2			
Custom support level 3			
General settings			
System info			
Battery			

Undo changes Write Save settings Load settings

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